




---

**INFINITE THERAPEUTICS**  
68 Route 125  
Kingston, New Hampshire 03848  
U.S.A.  
ph: (603) 347-3006 Fx: (603) 642-9291

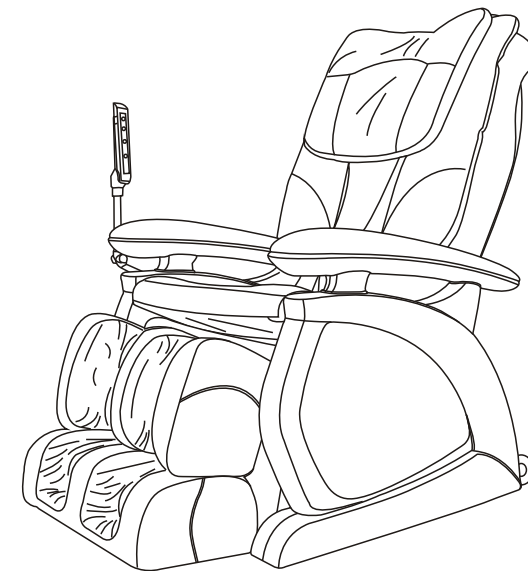

**Deluxe Multi-functional Massage Chair**

**IT-7800  
USER MANUAL**

Thank you for your purchase of this product. Before using, please read this USER MANUAL carefully, and pay special attention to the safety precaution to ensure proper operation and use.

Please keep this USER MANUAL properly for further reference.

## CONTENTS

SAFETY PRECAUTIONS .....	1
NAME AND FUNCTION OF EACH COMPONENT .....	3
Exterior structure .....	3
Interior structure .....	3
Controller .....	4
Instruction on each component of main controller .....	5
Adjustment on calf massages racks and backrest .....	6
PREPARATION BEFORE USING .....	8
Installation position .....	8
Installation method .....	8
Move method .....	11
Turn on power supply .....	11
PRODUCT STORAGE AND MAINTENANCE .....	12
TROUBLE SHOOTING .....	13
TECHNICAL SPECIFICATIONS .....	13

## TROUBLE SHOOTING

No.	Problem	Possible Cause	Solution
1	The sound “Xu Xu” , “Ta Ta” is heard.	It is a normal sound made by the working motor or the sound is produced when the massage ball is rubbed against the cloth.	It is normal and no need for treatment.
2	The machine can not be started up.	2.1 The power supply plug is detached.	2.1 Insert the power supply plug into the socket again.
		2.2 The power supply switch is not turned on.	2.2 Turn on the switch
		2.3 The fuse of massage chair blew out.	2.3 Replace a new one with the same mode
3	Backrest or legrest can not ascend or descend.	Over pressure is put on backrest or legrest.	Reduce the pressure on backrest or legrest.

If abnormal phenomenon still remains after above measures have been taken to the massage chair, please contact local distributor for repair and maintenance.

## TECHNICAL SPECIFICATIONS

Product name: Deluxe Multifunctional Massage Chair  
Product Model: IT-7800  
Rated voltage: 110-120V~  
Rated frequency: 50/60 Hz  
Rated power: 190W  
Safety design: Class

## ▶ PRODUCT STORAGE AND MAINTENANCE

### 1. Cleaning

- ▶ Before cleaning, make sure that the plug has been pulled out from the socket.
- ▶ Use a piece of cloth that has been dipped with cleaning agents to wipe the main unit of massage chair, and then use a piece of dry and soft cloth to wipe it.

#### Caution

- ▶ Do not use corrosive cleaning agents like benzene or thinner to clean the massage chair, otherwise, the color of upholstery will fade or the massage chair will crack or be eroded.
- ▶ Do not iron the upholstery.

### 2. Maintenance

- ▶ Make sure that the plug has been pulled out from the socket.
- ▶ Sweep off the dust. Keep the massage chair clean and away from damp places.
- ▶ If the massage chair is not to be used for a long period of time, please cover the massage chair with a dustproof cover (self provided).

#### Caution

- ▶ Do not store the product under direct sunlight or in places with high temperature. This may cause the color of the upholstery to fade.

### 3. controller

- ▶ Using wet cloth to clean the controller may cause failure of it, so it is advisable to use dry cloth to clean controller



### 4. Leather and lining



- ▶ Please use soft and dry cloth to clean the machine.
- ▶ Every two or three month, polish the lining with clean cloth that has been dipped with cleaning agents, and then clean it with dry and soft cloth after it dried.

## ▶ SAFETY PRECAUTIONS




### Safety Precautions

- ▶ Before using, please read this “SAFETY PRECAUTIONS” carefully to ensure proper operation.
- ▶ The following “SAFETY PRECAUTIONS” is of great help to your correctly use of this massage chair.
- ▶ To clearly indicate the degrees of danger and damage, the classes of accidents caused by incorrect use are classified into “WARNING” and “CAUTION”, instructions marked as followings should be strictly followed:

	<b>WARNING</b>	Improper operation may cause injury to users.
	<b>CAUTION</b>	Improper operation may cause users injury or product damage.

	Forbidden. Specific instruction will be indicated in the diagram drawings.(e.g, the left drawings means disassembly is not allowed.)
	Compulsory Specific instruction will be indicated in the diagram drawings.(e.g, the left drawings means to disconnect the power plug)

#### WARNING

	<p>We recommend that those with any of the following conditions should consult a doctor before using.</p> <div> <div> 1. Those who are using electronic medical instrument imbedded in the body, such as cardiac pacemaker etc.</div> <div> 4. Women who are pregnant or menstruating.</div> </div> <div> <div>2. Those that are being treated by doctors, especially those who feel uncomfortable.</div> <div> 5. Those with osteoporosis or fracture spine.</div> </div> <div> <div>3. Those who are malignant tumor patients, heart disease patients, acute patients.</div> <div> 6. Those with skin disease or those whose skins are injured.</div> </div> <div> <div>7. Those whose body temperature is over 38 degrees (Febrile Phase).</div> </div>
	<p>Do not allow the mentally disabled to use the chair alone. (It is OK under supervision)</p> <p>Do not allow children play on massage chair.</p> <p>Do not allow more than one person to use this massage chair at a time.</p> <p>If the power cord or power plug becomes damaged in any way you must disconnect from the mains supply immediately and consult an authorized service engineer for repair or replacement.</p> <p>Do not damage, process, excessively bend, pull hard, twist or knot power cord.</p> <p>Juveniles are not allowed to use this product.</p>
	<p>Anyone else, except maintenance personnel specified by our company, should not disassemble, maintain, or remodel this massage chair. If any problem, please contact local dealer immediately.</p>

## SAFETY PRECAUTIONS


### CAUTION

COMPULSARY	Please do not continuously use the massage chair over 20 minutes, and it is limited to massage a particular part of body in 5 minutes.
	Please put the massage chair on flat ground for use.
	When adjusting the legrest or backrest, please do not exert excessive force on it; in addition, keep children and pets well clear.
	When removing power plug, please hold the insulation plug, do not pull power cord.
	Please do not use the power supply that is beyond the specified electric voltage, as this may cause fire disaster or damage massage chair.
	Before using, please uncover the back cushion to check whether backrest or fabric is damaged; if damaged, please stop using this massage chair immediately, and contact franchiser for repair immediately.
	Please do not operate the controller with damp hands, and do not put heavy objects on controller.
	Only people less than 120kgs are acceptable.
	Please do not use other therapeutic equipments together with this massage chair.
	During use, if you feel uncomfortable, please stop use immediately and consult your doctor.
	This massage chair is designed and intended for non commercial household use only, do not use the massage chair for medical treatment.
	Do not use the massage chair in damp environment to avoid electric leakage or electric shock.
PROHIBITED	Always switch off power and remove the plug from the mains socket immediately after use and before cleaning.
	To avoid causing discomfort, do not use the massage chair immediately after meals.
	Do not fall into sleep on massage chair, when the massage chair is working.
	Do not use the massage chair when you are drunk or feel bad.
	If the upholstery becomes dirty, please use only known good quality leather cleaner, do not use chemical cleaners, petrol or detergents etc, and do not spray pesticide on it.
REMOVE PLUG	Do not sit on backrest, legrest, or armrest to avoid accidental injury or machine fault.
	Please do not smoke while using the massage chair.
REMOVE PLUG	Before maintenance, please pull out power plug. In addition, please do not insert or unplug the power plug with damp hands, to avoid electric shock.
	When the massage chair is not in use, please switch off power and remove the plug from socket. When power is off, please cut off power supply immediately and pull out the power plug from socket.

## PREPARATION BEFORE USING

### Move method

- ▶ The massage chair can be easily moved with the caster installed at the bottom of the chair.
- ▶ Ascend the backrest to the highest position.
- ▶ Turn off the power supply switch and pull out the plug.
- ▶ Raise the front part of the chair as shown on the picture below, and move the chair to a desirable place.

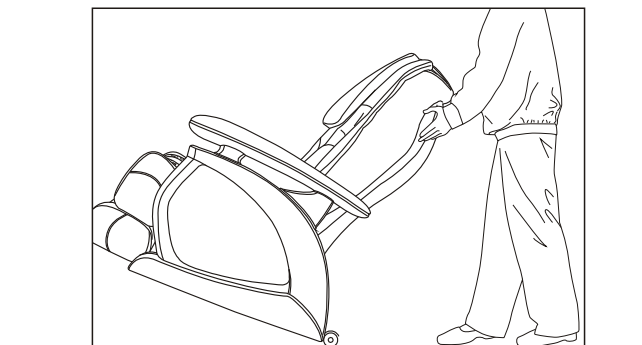

### Caution

- ▶ Do not move the chair when people sitting on it.
- ▶ When move the massage chair, please hold its base and backrest instead of its armrests (at least 4 people).

### Turn on power supply

- ▶ Put the power supply plug into a well grounding three-core power supply socket.
- ▶ Turn on the power supply switch installed at the lower part of backrest.


### Caution

- ▶ Before starting up the massage chair, please make sure the power supply cord and exposed electric wire have not been damaged.
- ▶ Before putting the plug into the power supply socket, please make sure the power supply switch of the product has been turned off.

## ► PREPARATION BEFORE USING


Step 5:

- Align the mounting holes of controller support with the two holes of armrest, and then fix them with two screws

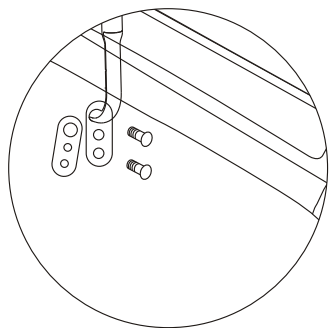


Step 6:

- Insert the plugs of controller and power supply cord to their corresponding sockets; Connect and switch on the power supply.

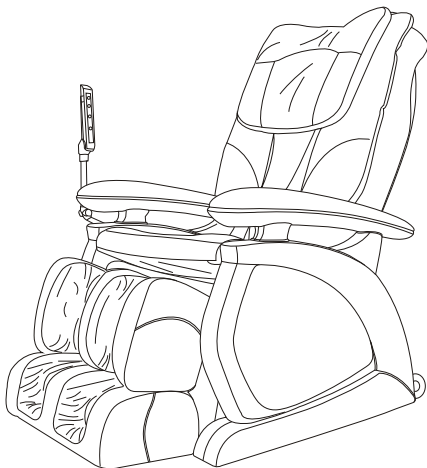

## ► NAME AND FUNCTION OF EACH COMPONENT

### Exterior structure

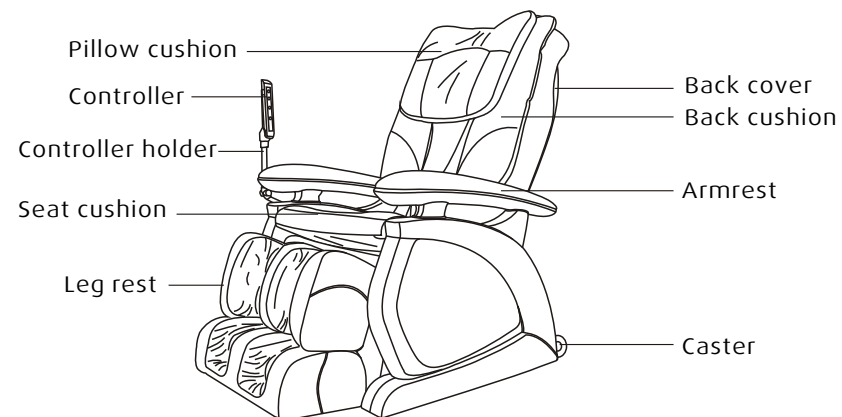


### Interior structure

Six roller massage mechanism, which travels from neck, shoulder to waist, it can give the massage modes of kneading, shiatsu, tapping and so on.

Airbag massage on the seat cushion


Airbag massage on the leg rest

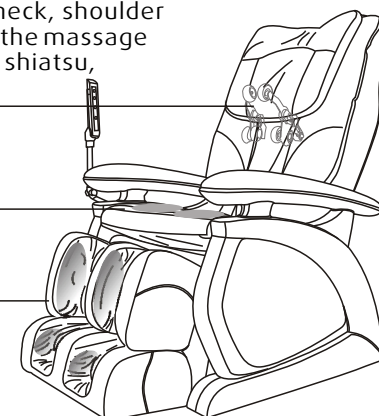

## ► NAME AND FUNCTION OF EACH COMPONENT

### Controller

The button of this controller is touch type, user can just gently touch the icons on controller to start each function.

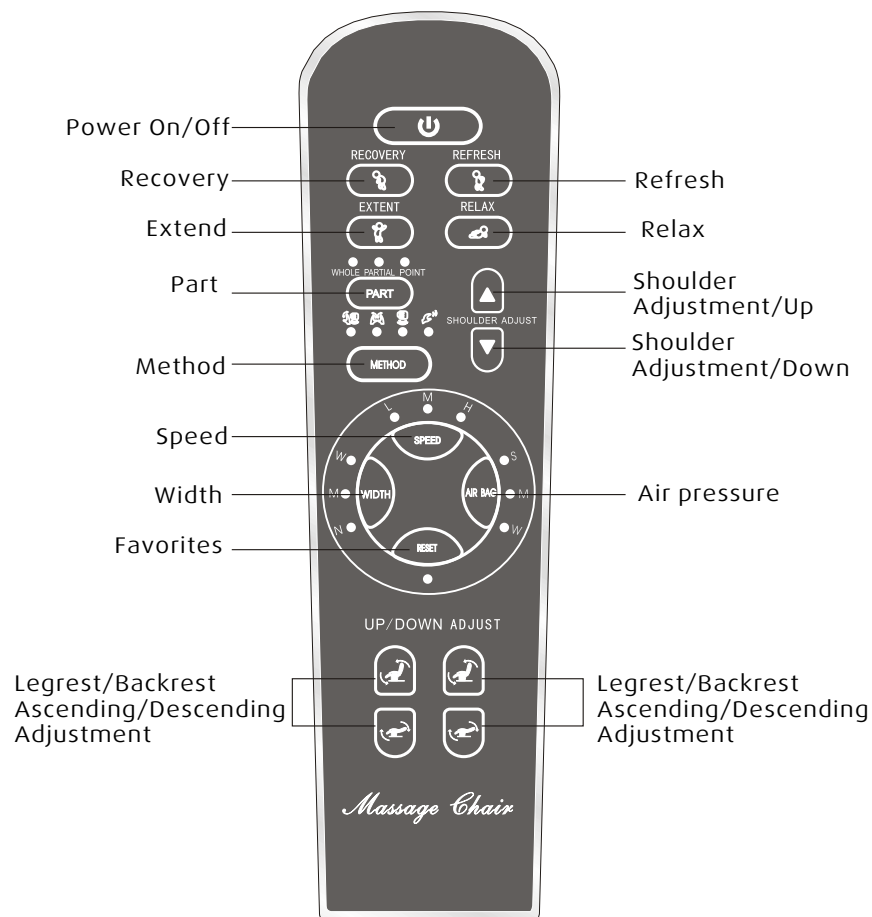


### ⚠ Caution

- In 2 seconds after turning on the power, please do not make the touch panel contact with any metal or human body directly, as this may affect the touch button's sensitivity.

## ► PREPARATION BEFORE USING


### Step 2:

- Raise up the backrest of massage chair at about 115°, then it will be locked when a reminding sound "Ka" is heard.

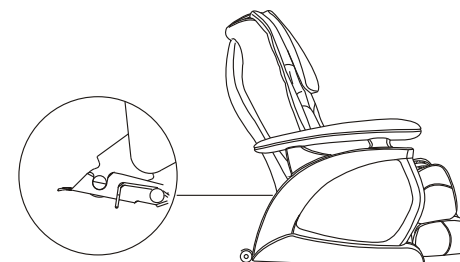


### Step 3:

- Put the seat cushion onto shelf, and connect the airbag tube as shown on the picture below.

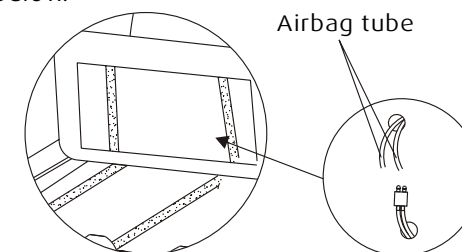

### Step 4


- Fix the back cushion onto backrest with zipper, and then fasten the big/small pillows with Velcro; Insert the plugs of backrest and seat cushion as shown on the picture below.

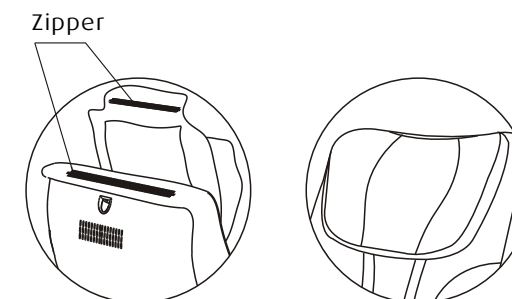

## ► PREPARATION BEFORE USING

### Installation position

- Please place the massage chair in an ample space: The horizontal interval between the backrest ascending to the highest point and descending to the lowest point should be 50 cm. The horizontal interval between the legrest ascending to the highest point and descending to the lowest point should be 40cm. To make sure more than 50 cm horizontal distances are left for the back part of chair, and more than 20 cm horizontal distances are left for the front part.
- Keep the massage chair more than 1 meter away from TV, radio, and some other audio-visual equipment to avoid signal interference.
- The floor may be worn by the massage chair, so it is advised to put some cushion material (for instance: a carpet) under the massage chair to protect the floor and reduce noises.

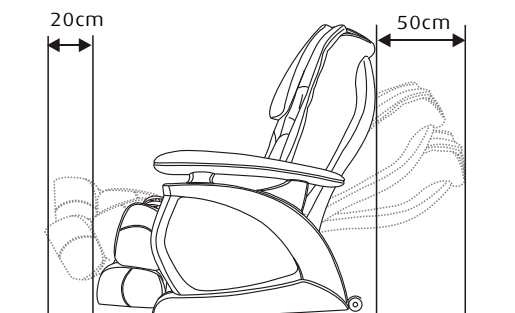


### Installation method

(More than 4 people are needed when move the massage chair)

#### Step 1:

- Open the carton, and take out the main unit and controller of the massage chair separately. Check whether the components are complete according to the packing list.

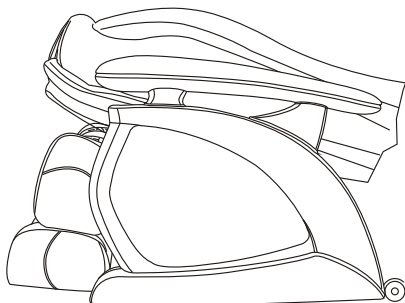

## ► NAME AND FUNCTION OF EACH COMPONENT

### Function Description on Controller

- Power On/Off:** press this button to turn on or off the whole unit. (The default massage time is 20minutes)
- When the indicator light of power icon is flashing, all the functional buttons are under non-operable state, press this button to turn on massage function; When power is on, the default massage mode is auto “Refresh”.
  - When the indicator light of power icon is constant on, press this button for 3 seconds to turn off all massage function, and then the massage chair will return to standby mode.
  - When the indicator light of power icon is off, all the function buttons on control panel will stop work.
- Recovery:** Press this button to turn on or off “recovery” auto massage function.
- RECOVERY**
- When the indicator light of this icon is off, press this button, the indicator light will be lighted up, and the “recovery” function is turned on; this automatic program adopts moderate strength massaging a large part of body.
  - When the indicator light of this icon is constant on, press this button, the indicator light will be off, and the “recovery” function is turned off; at this moment, the back massage unit is in halted state.
- Refresh:** Press this button to turn on or off “Refresh” auto massage function.
- REFRESH**
- When the indicator light of this icon is off, press this button, the indicator light will be lighted up, and the “refresh” function is turned on; this automatic program adopts moderate strength massaging a large part of body.
  - When the indicator light of this icon is constant on, press this button, the indicator light will be off, and the “refresh” function is turned off; at this moment, the back massage unit is in halted state.
- Extend:** Press this button to turn on or off “Extend” auto massage function.
- EXTENT**
- When the indicator light of this icon is off, press this button, the indicator light will be lighted up, and the “extend” function is turned on; this automatic program is a partial massage program mainly for massaging neck and waist.
  - When the indicator light of this icon is constant on, press this button, the indicator light will be off, and the “extend” function is turned off; at this moment, the back massage unit is in halted state.
- Relax:** Press this button to turn on or off “Relax” auto massage function.
- RELAX**
- When the indicator light of this icon is off, press this button, the indicator light will be lighted up, and the “Relax” function is turned on; this automatic program is a partial massage program mainly for massaging waist.
  - When the indicator light of this icon is constant on, press this button, the indicator light will be off, and the “relax” function is turned off; at this moment, the back massage unit is in halted state.
- Part:**
- WHOLE PARTIAL POINT**
- PART**
- Under manual massage mode, the indicator light of this icon is constant on, and this function button is at operable state; press this button to circularly demonstrate following modes: Whole, Partial and Point.
  - When the indicator light of this icon is off, this function button is at inoperative state.

## NAME AND FUNCTION OF EACH COMPONENT




### Method:

- ▶ When the indicator light of this icon is off, press this button to enter into manual massage mode.
- ▶ When the indicator light of this icon is on, press this button to choose following massaging method:  
Synchronization Knead Knock Pummel



### Shoulder Adjustment

- ▶ Under the “Point” massage mode, press “ ” and “ ” buttons to adjust the shoulder massage position.

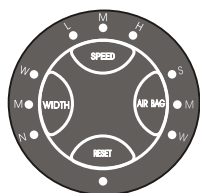

### Speed:

- ▶ Under manual massage mode, if the indicator light of this icon is constant on, this function button is at operable state; press this button to circularly choose following three speeds: low speed, medium speed and high speed;
- ▶ When the indicator light of this icon is off, this function button is at inoperative state.

### Width:

- ▶ Under manual massage mode, if the indicator light of this icon is constant on, this function button is at operable state; press this button to circularly choose following three widths: narrow, medium and wide;
- ▶ When the indicator light of this icon is off, this function button is at inoperative state.

### Airbag

- ▶ When the indicator light of this icon is off, if the massage function has been turn on, press this button to turn on airbag massage function.
- ▶ When the indicator light of this icon is constant on, press this button to circularly choose following 3 airbag massage strengths: weak strength, medium strength and strong strength; press this button again to turn off airbag massage function.

### Memory

- ▶ Press and hold this button for 3 seconds to turn off all massage function; after that, the massage chair will reset and return to standby mode.



Press “ ” or “ ” button to adjust legrest ascending and backrest descending.



Press “ ” or “ ” button to adjust legrest descending and backrest ascending.

## NAME AND FUNCTION OF EACH COMPONENT

### Legrest/Backrest Adjustment Method

Press “ ” or “ ” button on controller to adjust legrest/backrest to your desired positions. When they are adjusted to the limit positions, the massage chair gives out a series of reminding sound ( “Di, Di, Di...” ); During normal adjustment, the sound “Di Di . . Di Di . . .” will be heard.

Press “ ” or “ ” button on controller to adjust legrest/backrest to your desired positions. When they are adjusted to the limit positions, the massage chair gives out a series of reminding sound ( “Di, Di, Di...” ); During normal adjustment, the sound “Di Di ... Di Di ...” will be heard.

