




68 Route 125
Kingston, New Hampshire 03848
U.S.A.
T. (603) 347-6006 F.(603) 642-9291
www.infinitetherapeutics.com

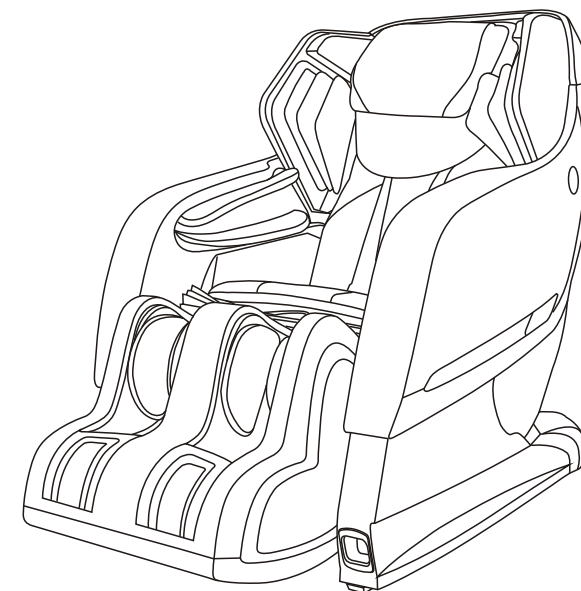

The New World of Massage

IYASHI
User Manual

Thank you for your purchase of this product. Before using, please read this USER MANUAL carefully, and pay special attention to the safety precautions to ensure proper operation and use.

Please keep this USER MANUAL for further reference.

CONTENTS

PRECAUTIONS	1
CONTROLLER INSTRUCTIONS OF EACH COMPONENT	3
External structure	3
Internal structure	3
Controller	4
Instruction of Controller	5
Instruction of Display screen menu	6
FUNCTIONS AND FEATURES	11
PREPARATION BEFORE USING	12
Installation Method	12
Moving method	16
Turn on power supply	16
Installation position	17
Floor protection	18
Massage chair first use	19
Before sitting in the chair	19
Auto Program	19
Legrest Extend and Retract	20
Blue tooth	20
After using	20
CONTROL VIA BLUE TOOTH	21
Software	21
PRODUCT STORAGE AND MAINTENANCE	22
TROUBLE SHOOTING	23
TECHNICAL SPECIFICATIONS	24

► TECHNICAL SPECIFICATIONS

Name: IYASHI

Rated voltage: 110V~

Rated frequency: 60 Hz



Rated power: 260W



Safety design: Class I

► PRECAUTIONS




The following safety precautions will help you to use chair correctly

- Before using, please read this “SAFETY PRECAUTIONS” carefully to ensure proper operation.
- The following safety precautions will help you to use chair correctly.
- To clearly indicate the degrees of danger and damage, the classes of accidents caused by incorrect use are classified into “WARNING” and “CAUTION”, instructions marked as followings should be strictly followed:

 WARNING	Improper operation may cause injury to users.
 CAUTION	Improper operation may cause users injury or product damage.




	Forbidden. Specific instruction will be indicated in the diagram drawings.(E.g, the left drawings means disassembly is not allowed.)
	Compulsory Specific instruction will be indicated in the diagram drawings.(e.g, the left drawings means to disconnect the power plug)

WARNING

 COMPULSARY	<p>We recommend that those with any of the following conditions should consult a doctor before using.</p> <ol style="list-style-type: none"> 1. Those who are using electronic medical instrument imbedded in the body, such as cardiac pacemaker etc. 2. Those that are being treated by doctors, especially those who feel uncomfortable. 3. Those who are malignant tumor patients, heart disease patients, acute patients. 4. Women who are pregnant or menstruating. 5. Those with osteoporosis or fracture spine. 6. Those with skin disease or those whose skins are injured. 7. Those whose body temperature is over 38 degrees (Febrile Phase).
 PROHIBITED	<p>Do not allow the mentally disabled to use the chair alone. (It is OK under supervision)</p> <p>Do not allow children play on massage chair.</p> <p>Do not allow more than one person to use this massage chair at a time.</p> <p>If the power cord or power plug becomes damaged in any way you must disconnect from the mains supply immediately and consult an authorized service engineer for repair or replacement.</p> <p>Do not damage, process, excessively bend, pull hard, twist or knot power cord.</p> <p>Juveniles are not allowed to use this product.</p>
 NO DISASSEMBLY	<p>Anyone else, except maintenance personnel specified by our company, should not disassemble, maintain, or remodel this massage chair. If any problem, please contact local dealer immediately.</p>

PRECAUTIONS

CAUTION

 COMPULSARY	<ul style="list-style-type: none"> ◆ Please do not use the massage chair continuously for over 20 minutes and do not massage a particular body part for more than 5 minutes. ◆ Please put the massage chair on flat ground for use. ◆ When adjusting the legrest or backrest, please do not exert excessive force on it; in addition, keep children and pets well clear. ◆ When removing power plug, please hold the insulation plug, do not pull power cord. ◆ Please do not use the power supply that is beyond the specified electric voltage, as this may cause fire disaster or damage massage chair. ◆ Before using, please uncover the back cushion to check whether backrest or fabric is damaged; if damaged, please stop using this massage chair immediately, and contact direct people to tech support for repair immediately. ◆ Please do not operate the controller with damp hands, and do not put heavy objects on controller. ◆ Only people less than 265 lbs are acceptable. ◆ Please do not use other therapeutic equipments together with this massage chair. ◆ During use, if you feel uncomfortable, please stop use immediately and consult your doctor. ◆ This massage chair is designed and intended for non commercial household use only, do not use the massage chair for medical treatment. ◆ The massage chair is equipped with heating function, so people who are insensitive to heat must use it carefully. ◆ Always switch off power and remove the plug from the mains socket immediately after use and before cleaning.
 PROHIBITED	<ul style="list-style-type: none"> ◆ To avoid causing discomfort, do not use the massage chair immediately after meals. ◆ Do not fall into sleep on massage chair, when the massage chair is working. ◆ Do not use the massage chair when you are drunk or feel bad. ◆ If the upholstery becomes dirty, please use only known good quality leather cleaner, do not use chemical cleaners, petrol or detergents etc, and do not spray pesticide on it. ◆ Do not sit on backrest, legrest, or armrest to avoid accidental injury or machine fault. ◆ Please do not smoke while using the massage chair.
 REMOVE PLUG	<ul style="list-style-type: none"> ◆ Before maintenance, please pull out power plug. In addition, please do not insert or unplug the power plug with damp hands, to avoid electric shock. ◆ When the massage chair is not in use, please switch off power and remove the plug from socket. When power is off, please cut off power supply immediately and pull out the power plug from socket.

TROUBLE SHOOTING

No.	Problem	Possible Cause	Solution
1	The sound “Shu Shu”, “Ta Ta” is heard.	It is a normal sound made by the working motor or the sound is produced when the massage ball is rubbed against the cloth cover.	It is normal and there is no need for repair
2	The machine can not be started up.	The power supply plug is detached.	Insert the power supply plug into the socket again.
		The power supply switch is not turned on.	Turn on the switch
3	No sound from the speaker	The fuse of massage chair blew out.	Replace fuse with new fuse of same type
4	Backrest or legrest can not ascend or descend.	The volume is turned to lowest.	Turn the volume up
5	The airbag installed at seat cushion or legrest does not work.	Over pressure is put on backrest or legrest.	Reduce the pressure on backrest or legrest.
		There could be a pinched or blocked air tube.	To clear the air tube

If abnormal phenomenon still remains after above measures have been taken to the massage chair, please contact local distributor for repair and maintenance.

Note: Do not dismantle the product by yourself, and our company shall shoulder no responsibility to the damage of the product or the injury of people caused by improper use! Thanks your cooperation.

▶ PRODUCT STORAGE AND MAINTENANCE

Maintenance

- ▶ Clean plastic parts and components with soft cotton cloth and mild cleaner. Always test cleaner in non-conspicuous area.
- ▶ Use dry soft cotton cloth to wipe the controller and power supply box.
- ▶ All Synthetic Leather should be wiped with a leather cleaner.

⚠ Caution

- ▶ Make sure to pull out the power supply plug first before maintenance. Do not use wet hands to pull out the plug to avoid electric shock. When clean the back cover, please be cautious of detergent entering into the power supply box!
- ▶ Do not iron the upholstery.

Storage method

- ▶ Wipe off dust and dirt from the product after use, if the product is not to be used for a long period of time, please cover the product with a dust cover (self provided).

⚠ Caution

- ▶ Do not store the product under direct sunlight or in places of high temperature. This may cause the color of the upholstery to fade.

Please Remember

During daily use, please check whether the following symptoms exist:


- ▶ Scorched smell exists.
- ▶ Moving power cord causes power to shut off.
- ▶ Other abnormal symptom

⚠ Caution


- ▶ Please stop using the massage chair to avoid breakdowns and accidents occurring if any above symptom exists.
- ▶ Turn off the power supply switch and pull out the power supply plug, and then contact the local distributors or headquarters for repair and maintenance.
- ▶ Except all the normal operation and maintenance mentioned above, any other repair and maintenance can only be operated by our authorized dealer or service agent.

▶ NAME AND FUNCTION OF EACH COMPONENT

External structure

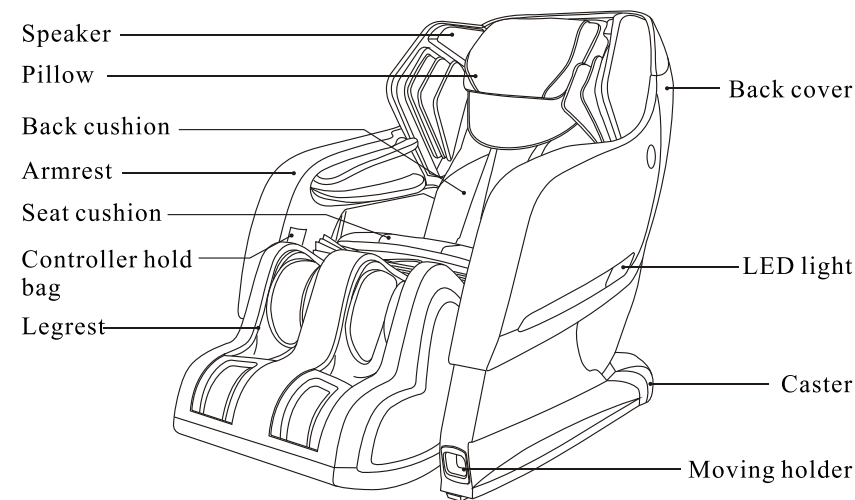


Internal structure

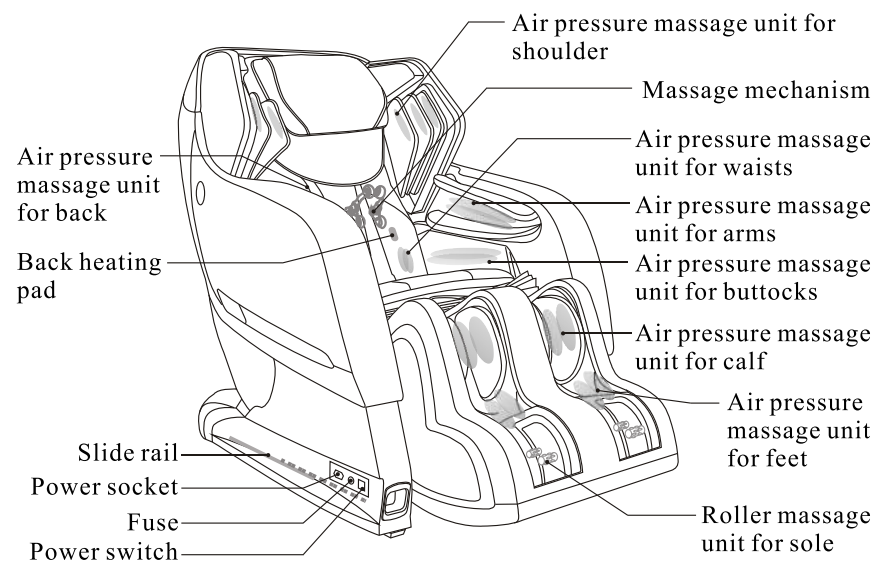

► NAME AND FUNCTION OF EACH COMPONENT

Controller

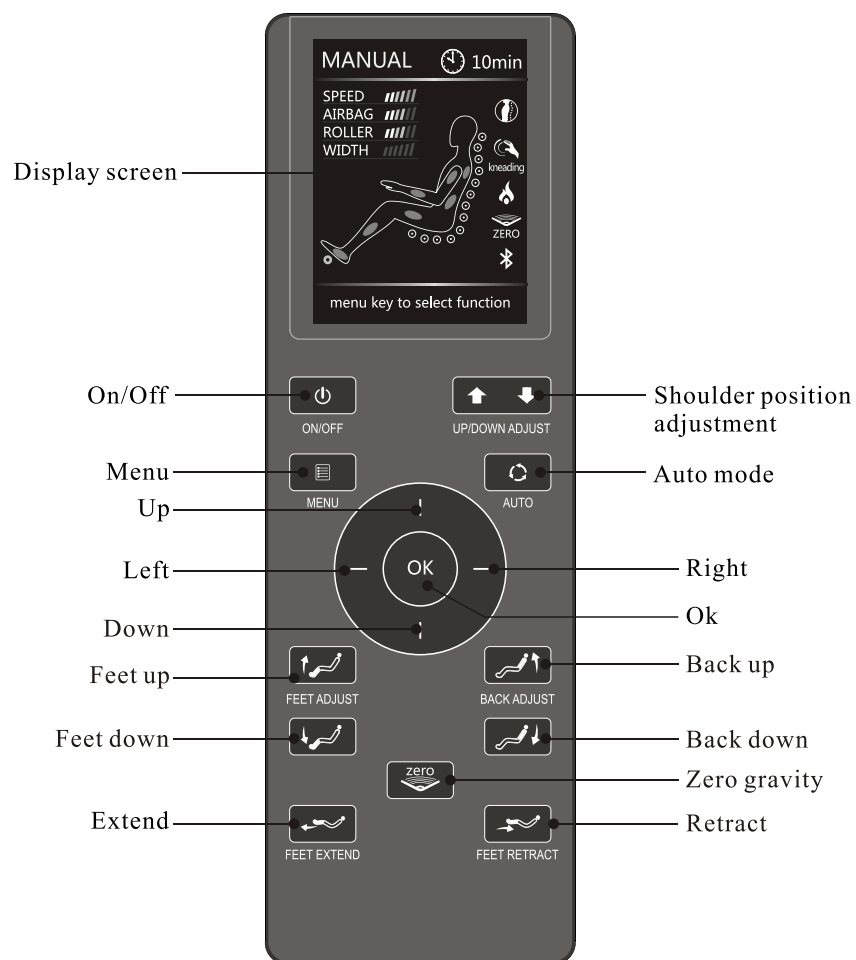



► CONTROL VIA BLUE TOOTH


Software

- Download software from the Apple App Store or Google Play Store, depending on device.

► PREPARATION BEFORE USING

Legrest Extend and Retract

- When the legrest in original state, in order to satisfy the most suitable massage position, press  or  to manually adjust the relative position between the footrest frame and leg.

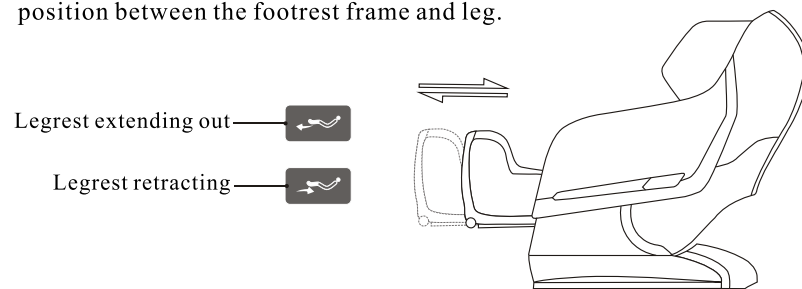

- Legrest can be extended or retracted automatically or manually according to different position and height, because of sensor inside

WARNING

- Do not play in the chair and conduct improper operation.
- Before adjusting the angle between calf and backrest parts you need to make sure no obstacles is around.
- Do not insert leg, foot or other goods into chair's clearance when leg parts extend or reset.
- Do not get down from the chair when the chair is in the reset state.

Blue tooth

- This device can play audio via Bluetooth. Open Bluetooth from the Bluetooth audio compatible device and pair with Rt8600.
- There is extra audio inputting cable for other audio devices next to the controller socket. (optional)

After using

- Please power off the chair and unplug the chair power cable from the power supply socket after using the massage chair.
- Store controller inside pocket on chair.
- Please power off the chair and unplug the chair from the power supply socket when the chair is not in use, to prevent children from starting chair accidentally.

► CONTROLLER INSTRUCTIONS OF EACH COMPONENT

Instruction of Controller

On/Off

- When power is on, click this button, display screen will light up, then click up/down button to choose the auto mode, manual mode, air pressure massage, heat, roller massage, and set menu.
- Click this button again to turn off all functions. The default working time is 20 minutes

Menu

- Click this button to choose auto mode, manual mode, air pressure massage, heat, roller massage, and set menu.

Shoulder position adjustment

- Under manual and point mode, press UP/DOWN to adjust the position of point massage.

Auto mode

- Click this button to choose one of auto modes, such as “RECOVERY, EXTEND, RELAX, REFRESH, UPPER BODY AUTO, and LOWER BODY AUTO”



Direction button

- In the menu and auto mode, click up, down, left, right button to choose the concerned function, then click “OK” to confirm.

Zero gravity

- Click this button to turn On/Off zero gravity, then the relevant icon in the screen will light up. After adjusting the backrest and legrest position manually, zero gravity will be off.

Feet up

- Press this button, the legrest will be moved up to the top position, releasing this button, it will stop.

Feet down

- Press this button, the legrest will move down to the lowest position, releasing this button, it will stop.

▶ CONTROLLER INSTRUCTIONS OF EACH COMPONENT



Back up

▶ Press this button, the backrest will be moved up to the original position, release this button, it will stop.



Back down

▶ Press this button, the Backrest will be moved down to the lowest position, release this button, it will stop.



Legrest extend

▶ Press this button, the legrest will be extended to the longest position, releasing this button, it will stop.



Legrest retract

▶ Press this button, the legrest will be retracted to the original position, releasing this button, it will stop.




WARNING

▶ Press power button after use it to make the chair in reset state. Please press any buttons in the remote controller to stop reset. Press the power button again to continue the operation or enter into massage state.

Display Screen Instructions

▶ Auto mode

When power is on, click “ON/OFF” button, screen will light up, then Click the “AUTO” button to start one of six auto modes, such as “RECOVERY, EXTEND, RELAX, REFRESH, UPPER BODY AUTO, and LOWER BODY AUTO”

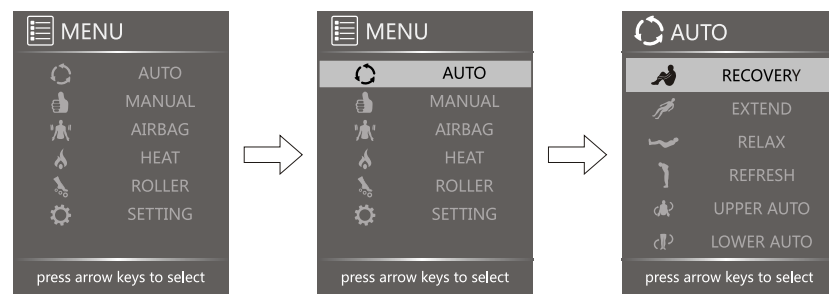

Recovery

For after a hard days work, to relieve tiredness and recover vigour.


Extend

Great for people who are in a sitting position all day, to stretch fully and recover flexibility.

▶ PREPARATION BEFORE USING

Massage chair first use

- ▶ Before first use, please make sure there is enough space around the chair. Then use the controller to recline down the chair backrest to the limit position, rise up the legrest to the highest position and stretch out to the limit length.
- ▶ Testing all functions work normally, then power off the chair, return back to original position.
(How to select the program, please refer to “Controller instruction” and “display screen menu instruction”)

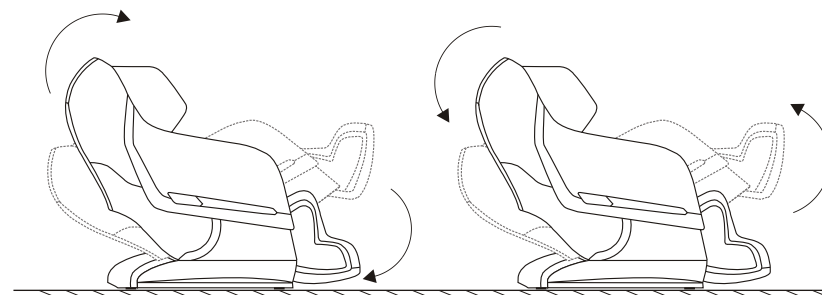

Before sitting in the chair

- ▶ Make sure the legrest is installed correctly.
- ▶ Make sure the chair is in original position, if any part is not in initial position, please power on/off again to make the whole chair go back to original position.



Caution

- ▶ If the legrest rises up, please do not sit on the chair.
- ▶ Do not put any sudden heavy load on the legrest or it may damage the mechanism.


Auto Program

- ▶ According to your body status and massage time to choose any massage program among below six AUTO programs as RECOVERY, EXTEND, RELAX, REFRESH, UPPER BODY, LOWER BODY. (Please refer to AUTO program function instruction.)

► PREPARATION BEFORE USING

Floor protection

- Please put carpet or other pad under the massage chair, in order to avoid any markings or impressions on the floor.

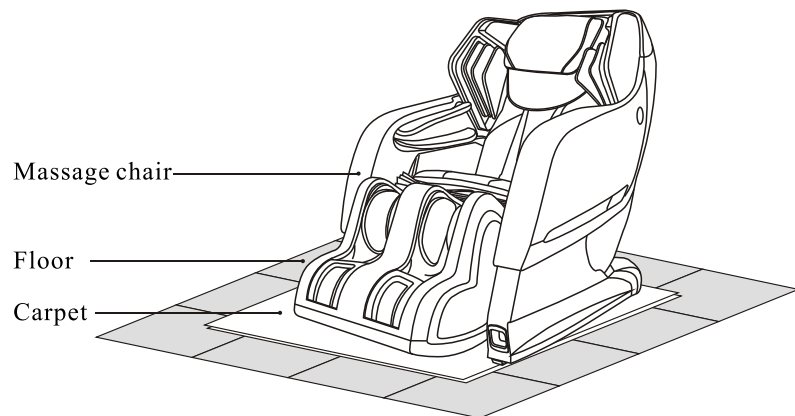

Surrounding environment



Keep the massage chair away from damp places (for example: bathroom) to avoid electric leakage.



Keep the massage chair away from high temperature sources (for example: stove) to avoid fire or damage to leather material.

Grounding

This massage chair belongs to Level One electrical appliance, make sure to use three-core power supply socket and to guarantee the ground wire connected to the power supply socket has obtained good ground connection to avoid electric leakage, electric shock and some other negative effect during use.

► CONTROLLER INSTRUCTIONS OF EACH COMPONENT

Relax

For early morning or noon leisure massage, to generate muscle energy.

Refresh


For relieving the pains in neck, back and waist, combining with semi-auto mode, it can ease the pain effectively with focusing massage.

Upper/Lower auto

For massaging different part of the body.


► Manual mode

Click “MENU”, choosing “MANUAL” to start manual program, then click up/down to choose: mode, part, width and speed.

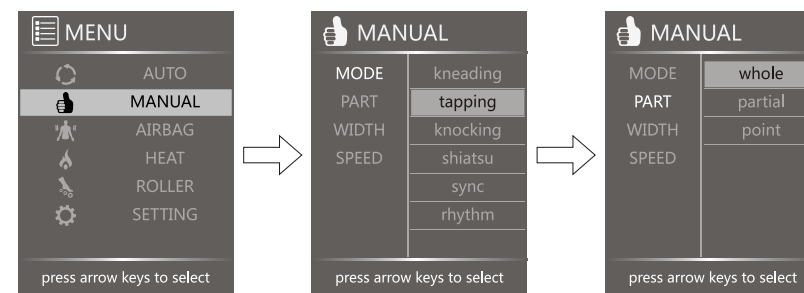

Mode: Click right button, then click up and down to circularly choose kneading, tapping, knocking, shiatsu, sync and rhythm massage.

Part: Click right button, then click up and down to circularly choose full, partial and point massage.

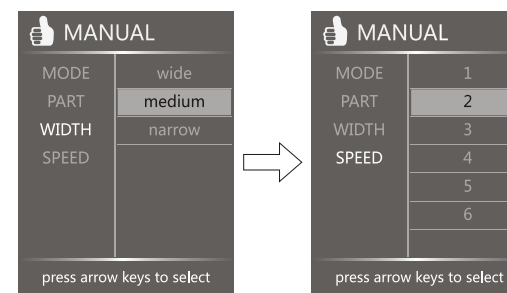


Width: Click right button, then click up and down to circularly choose wide, medium, narrow to adjust width of massage.

Speed: Click right button, then click up and down to circularly choose 1-6 levels of massage speed.

▶ CONTROLLER INSTRUCTIONS OF EACH COMPONENT

▶ Airbag

Click “MENU”, choosing “AIRBAG”, then click up/down key to choose: Part and Force.

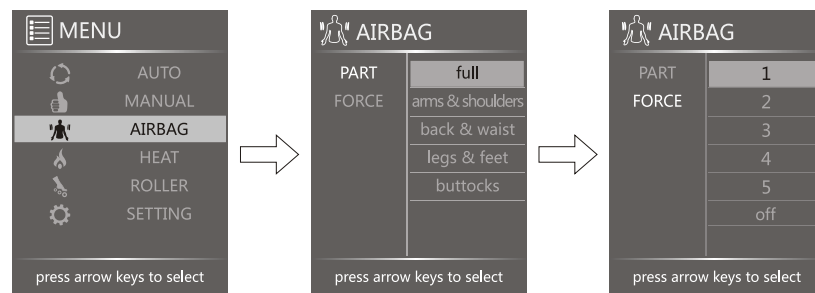


Part: Click right button, then click up and down to circularly choose full, arms & shoulders, back & waist, legs & feet, and buttocks

Force: Click right button, then click up and down to circularly choose 1-5 levels air strength.

▶ Heat

Click “MENU”, choosing “HEAT”, then click right to choose on or off.

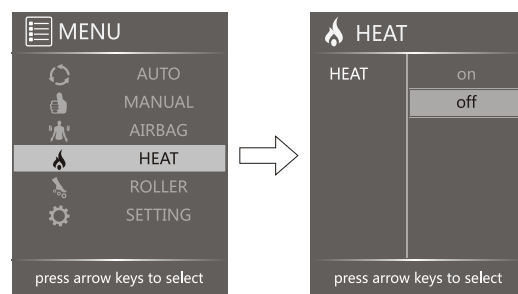


▶ PREPARATION BEFORE USING


Installation position

Safety distance for the periphery of massage chair.

- ▶ Please make sure there is adequate space for installation of this product:
Back: minimum 10cm from the wall or any other blocks;
Front: minimum 60cm from any blocks;
- ▶ In order to prevent signal interference, please keep at least 1m away from the TV, radio and other video and audio devices.

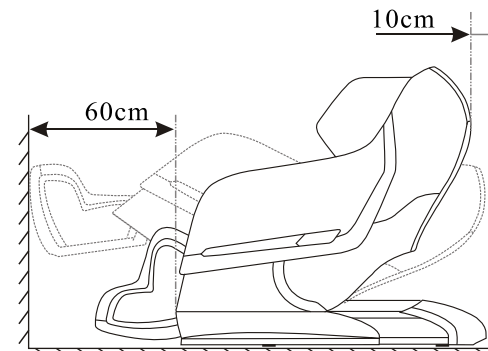

- ▶ When installation, the distance between the power socket and the power switch must be within 1.5m.

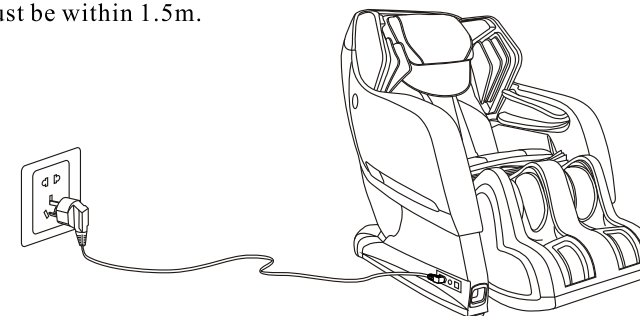


⚠ Caution

- ▶ Make sure the distance between the power socket and the power switch is less than 1.5m to avoid damage the power cables when the chair reclines up and down.
- ▶ Before using please insure cables are away from chair to avoid twisting and damage.
- ▶ When the whole chair makes parallel movement forward or return back to original position, please check first whether there are kids, pets or some other barriers to avoid accidents.

► PREPARATION BEFORE USING

Moving method

- There are moving rollers at the bottom of the chair.
- Raise the back to the highest point.
- Switch off and unplug the chair.
- Lift the front part of the chair as showed in the figure(one person raise the front part via holding moving holder, another person push the chair when holding the top of the chair) , move the chair by the rollers.

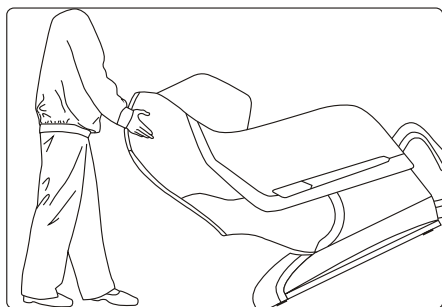


⚠ Caution

- Don't move it when there is anyone in the chair.
- Don't use the armrest while moving the chair, only hold the moving holder in the back and front

Turn on power supply

- Plug the power cord into three core socket with well grounding.
- Power on the chair by pressing the button at the lower part of the exterior of the right armrest .

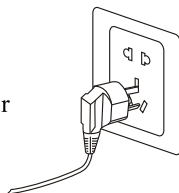


⚠ Caution

- Before switching on the chair, please make sure there is no damage on the power wire and other exposed wires.
- Make sure the power switch is off before the chair is connected to electricity.

► CONTROLLER INSTRUCTIONS OF EACH COMPONENT

► Roller

Click “MENU” , choosing “ROLLER” , then click up and down to circularly choose 1-3 levels of speed and off.

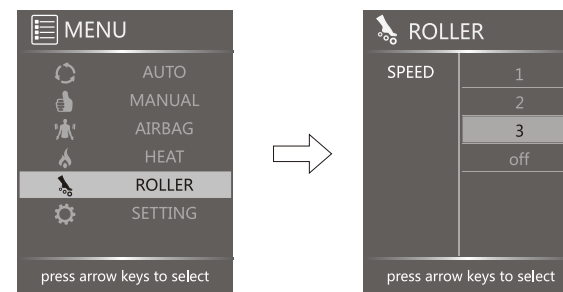


⚠ Caution

- Under the air massage of full body, the roller speed can not be adjusted.

► Set Menu

Press “Menu” button, select “Setting” to enter into Setting menu. Press “Up” or “Down” button to choose Time and Bluetooth setting.

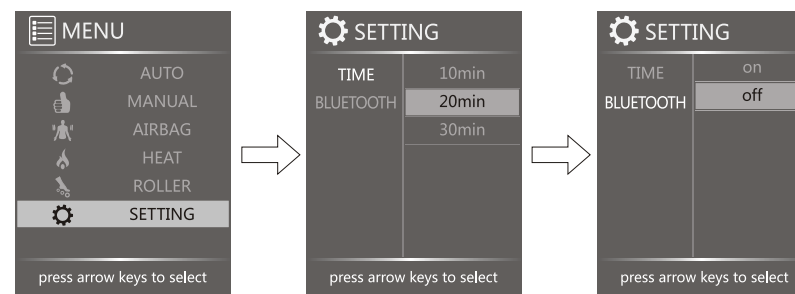


Time: Setting working time, firstly, press “Right” button, and then press the “Up” or “Down” button to adjust 10, 20, 30 minutes, and the default working time is 20 minutes.

Bluetooth: Setting “Bluetooth” , press the “Up” or “Down” button to open or close this function.


▶ CONTROLLER INSTRUCTIONS OF EACH COMPONENT

Note:

1. Under auto mode or manual mode, it will go back to home page as below drawing 10 seconds after without any operating.

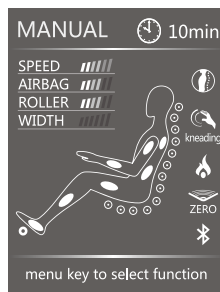

Home page in manual mode

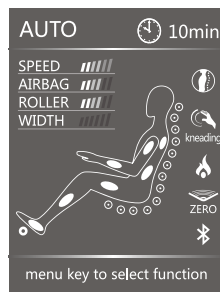


Home page in auto mode

⚠ Caution

- ▶ During working, click “MENU” twice, it will go back to home page immediately.
- ▶ In the home page, under manual mode or auto mode, click the left and right button to adjust air pressure intensity; under manual mode, click up and down, to adjust massage wheels speed.
- ▶ Click “AUTO” to choose auto program directly.

2. Before running auto program, it will inspect shoulder position first. Except “Lower Auto” program

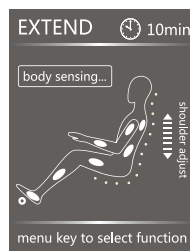


⚠ Caution

- ▶ Please do not disassemble the product for any reason.
- ▶ Please do not spill water and other liquid into the controller to avoid malfunction of keys and error code.
- ▶ Please do not put heavy things on the controller or power cable.

▶ PREPARATION BEFORE USING


Step 5 Installation of controller unit

- ▶ Connect the data line of the controller to the port on the inner side of right armrest (pay attention to the convex-concave between connector and port); then fix the screw cap. (see the figure below)

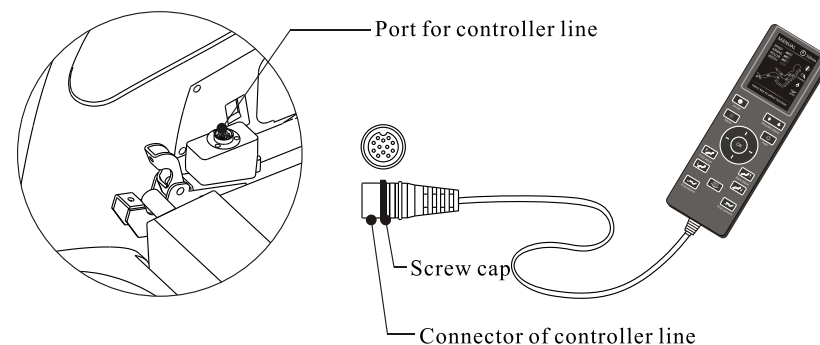

Step 6 Other Accessories


- ▶ Connect the power wire connector to the port on the outer side of right armrest;
- ▶ Switch on the chair, and use the controller to raise the backframe to normal position.

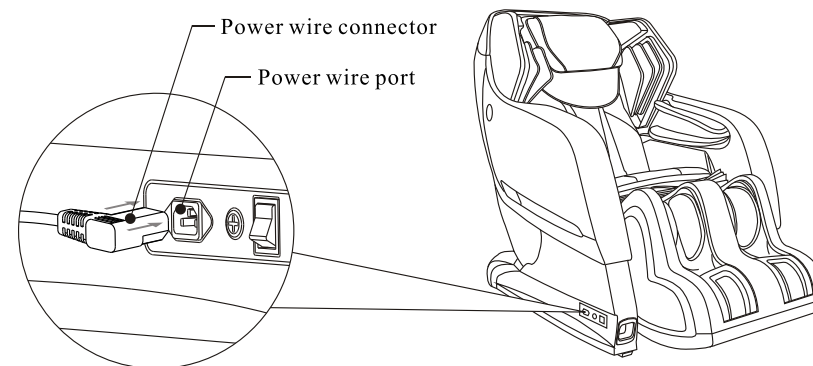


► PREPARATION BEFORE USING

Step 4 Installation of legrest unit


- Install the inner cover and outer cover of leg frame onto the connecting shaft.

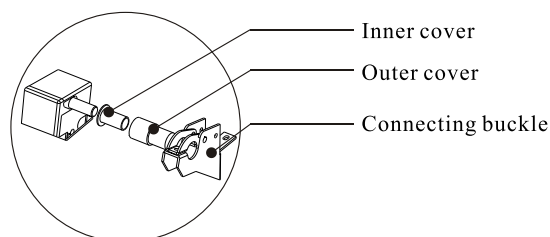

- Remove the screw on leg connecting buckle of the frame and open the buckle upwards, then put the leg units into the buckle groove of the frame. Close the buckle and fix it with bolt.

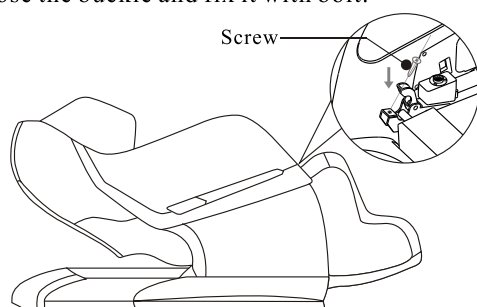


- Lift the leg units, plug the three connectors from leg frame into the three ports under the seat, and make sure they are well connected. (see the fig below)
- Connect the wrinkle cover of the leg massage units with the magic tape on the seat cloth.

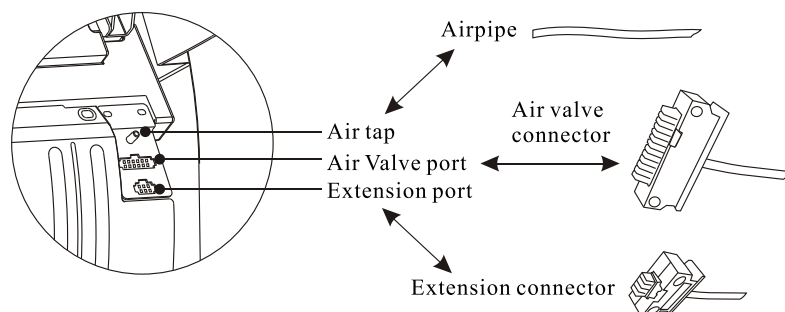

► FUNCTIONS AND FEATURES

1. Automatic Moving

There is a moving rail under the seat, The chair can move forward by 30cm upon clicking one button, saving room when not in use.

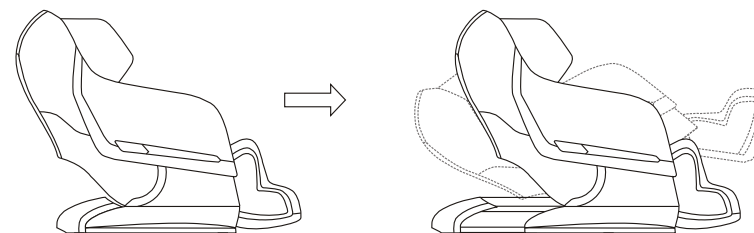

2. Zero Gravity

This chair uses the NASA zero gravity principle, keeping the legs higher than the heart, imitating weightlessness. This position is known to increase comfort and release stress.

3. Four wheel massage mechanism

The innovative four wheel massage mechanism and traveling structure perfectly match ergonomic human engineering.

4. Manual/auto legrest extension

Auto/ manual leg extension function. maximum 26cm to meet different users "height"

5. Air pressure massage

Air pressure massage for back, waist, shoulder, arm, buttock, calf, and feet.

6. Roller massage for sole

Double row rollers massage on foot conduct an all-around shiatsu massage onto the sole.

7. Diversified massage modes

Diversified massage modes: knocking, kneading/knocking, kneading, shiatsu, which act as a professional massagist

8. L shape super long massage track

► PREPARATION BEFORE USING


Installation Method

Step 1: Get out each part of the chair

- Unpack the cartons, remove speaker units, accessories box, and so on.
- Check all accessories with packing list.

Step 2 air bag massage unit for shoulder and speaker installation


- Connect outlet wire from air bag massage unit for shoulder and speaker with side ports on the upper back;
- Match the two grooves from air bag massage unit for shoulder and speaker to two connecting poles on the back, then insert and fix them by 4 screws. (It is the same installation steps for the other side)

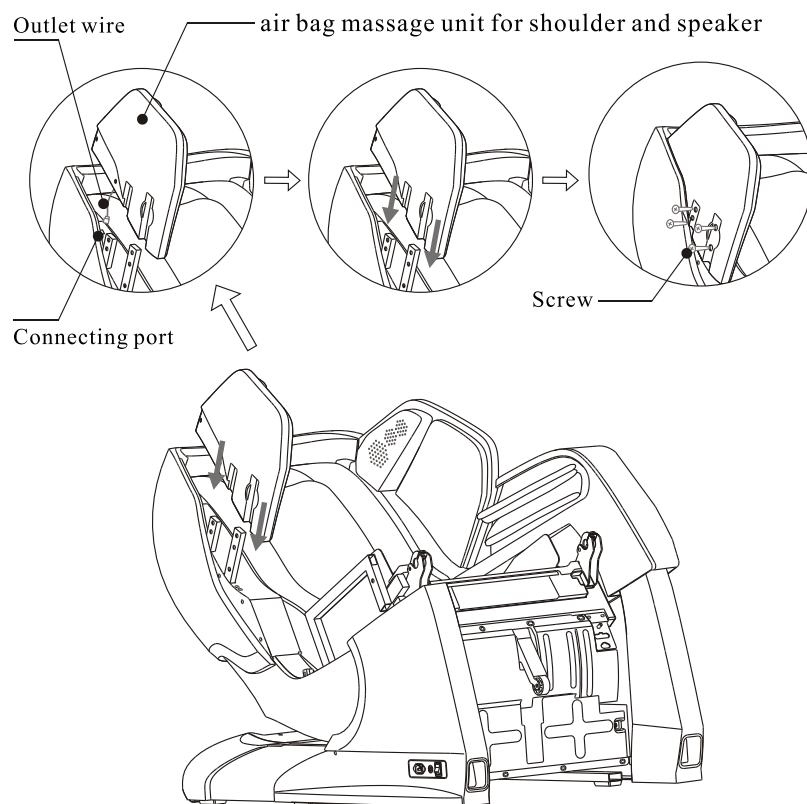

► PREPARATION BEFORE USING

Step 3 Installation of armrest units

- Match the air valve, color light, airpipe plugs on armrest to the side ports on lower part of the armrest frame. (follow the sequence in the diagram below)

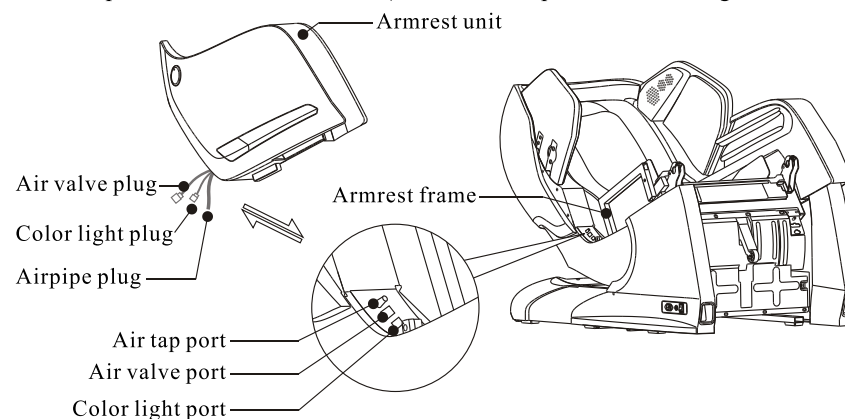

- After the cable from armrest are well connected, push downward after aiming the inner groove of armrest unit to armrest frame, then armrest unit will be well connected into the armrest frame, and fix them by a screw (a). (It is the same installation method for the other side)
- After the screw is tightly fixed, put the cover on (left and right are different), then fix the cover (b) by clockwise rotating.
- Use another screw to fix armrest from the interior (c).

